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STERILE HYPODERMIC SINGLE USE SYRINGE WITH NEEDLE

1. Intended use

Sterile hypodermic single use syringe with needle is a device intended for subcutaneous, intradermal, intramuscular and intravenous injections and withdrawals of body fluids by qualified and trained medical personnel.

2. Instructions for injection

1. Injection should be carried out in accordance with the requirements of its technique and aseptic rules in order to prevent contamination.
2. Before use, inspect tight individual package for integrity and its expiry date. Do not use syringe if the individual packaging is damaged or expired.
3. Nominal capacity, needle size and length are determined by the amount of drug to be injected and the type of injection (subcutaneous, intradermal, intramuscular or intravenous).
4. Wash hands thoroughly with detergent, treat with antiseptic. Put on sterile medical gloves.
5. Open the sterile individual package and remove the syringe with needle from the package on the side of plunger. Remove carefully the needle cap from the needle.

Note. If the needle is packed next to the syringe, put the needle firmly to the tip of the syringe and remove needle cap.

6. Draw up the required amount of injection fluid into the syringe, place syringe with needle vertically, remove the residual air by pushing the plunger.
7. Treat injection site with antiseptic.
8. Give injection into the prepared site according to the injection technique, depending on the type of injection (subcutaneous, intradermal, intramuscular or intravenous).

Note. When using syringe with two needles, use one needle for taking medicines, the other for injection.

9. Pull out the syringe with needle, gently squeeze the injection site with an antiseptic wipe. Blood droplets are allowed to appear around the needle injection.
10. In case of fluid suction from the body cavities, open sterile individual package and remove the syringe with needle from the package on the side of plunger. Remove carefully the needle cap from the needle. Carry out suction.
11. In case of body cavities irrigation, open sterile individual packaging and remove the syringe with a needle from the package on the side of plunger. Carefully remove needle cap from the needle. Fill the syringe with washing solution and hold washing.

3. Utilization

Syringe and needle should be utilized in accordance with waste disposal requirements after use. Use non-puncture container to collect and utilize used syringes and needles.

4. Warnings!

12. Do not use if package is damaged or opened.
13. For single use only. Re-use might cause infection.
14. Do not use after expiry date.
15. Not use with high viscosity materials.

Note. «UMEDO GROUP LLC» is not responsible for injuries or damages resulting from the improper use of this product.

5. Sterilization

Sterilized using ethylene oxide.

6. Storage conditions

Store in tight individual package, in dry place with temperature from +5 °C to +40 °C and relative air humidity no more than 80 %.

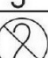
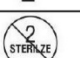
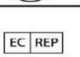
Keep out of the reach of children.

7. Shelf life

5 years unless the package is damaged.

Attention!

Syringe name, needle size, batch number, date of manufacture and expiry date are indicated on the individual, group and transport packaging.

Symbols and their description			
	Symbol «Manufacturers»		Symbol «Date of manufacture»
	Symbol «Catalogue number»		Symbol «Batch code»
	Symbol «Use-by date»	SIZE	Symbol «Needle size»
	Symbol «Do not use if package is damaged or opened!»		Symbol indicating sterilization using ethylene oxide
	Symbol «Non-pyrogenic»		Symbol «Temperature limit»
	Symbol «Keep dry»		Symbol «Keep away from sunlight»
	Symbol «Re-use is forbidden»		Symbol «Do not re-sterilize»
	Symbol «Special disposal. Dispose of in accordance with waste disposal requirements after use»		Symbol «Authorized representative in the European Community»
	Symbol «Conformity to the Technical Regulation»		Symbol «European Union conformity mark»

Manufacturer

«UMEDO GROUP» LLC, 62B, Vatutin general Str.
Makariv town, Makariv district,
Kyiv region, 08000, Ukraine

Tel. 0-800-502-363;
e-mail: info@umedogroup.com;
web: www.umedogroup.com

UA.TR.101

MedLine Devices s.r.o.
Francouzská 836/31, Vinohrady, 120 00, Praha,
Czech Republic

Date last viewed October 07, 2021
Ver. 04